REF

BAR022

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INSTRUCTION OF USE

1. General warnings

- For correct use of the device, refer to this manual
- The product must only be used by qualified personnel
- Check that the product is delivered to you intact and keep it packed away from heat sources
- The useful life of the device is determined by the wear of non-repairable and / or replaceable parts
- Store in dry places
- Always pay attention to the presence of children

2. Before each use

- Always check the state of wear of the parts in order to guarantee use of the device in total safety for people and things

3. Warnings for use

- Use the seat belt when necessary
- Periodically check the stability and robustness of the product. Replace it immediately in the presence of breakages.
- Do not use with patients whose weight exceeds that indicated. • In case of doubts about use, about product maintenance, please contact your dealer • To avoid any risks, keep the device out of the reach of children.

4. Instructions

- Open the stretcher • Correctly move the patient onto the stretcher • The presence of handles it allows the patient to be lifted by grasping the stretcher from several sides.

5. Maintenance

The devices at the time of placing on the market are carefully checked and bearing the CE mark. In case of repairs, only spare parts ed must be used original accessories.

6. Cleaning and disinfection

Use water and a mild disinfectant; dry away from direct sunlight.

Absolutely avoid using acidic, alkaline or solvent products such as acetone or the thinner.

7. Disposal conditions

In case of disposal of the device, never use the normal waste disposal systems urban solids. Instead, it is recommended to dispose of the device through the common islands municipal ecological systems for the planned recycling operations of the materials used.

It is necessary to report any serious accident occurring in relation to the DM provided by PVS SPA to the Manufacturer and the competent authorities

ZHANGJIAGANG HERUI MEDICAL CO LT CHINA

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PVS SpA