Production record certificate No: 苏锡
食药监械生产备 20150001 号

Product record certificate No:

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Technical requirement No:

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Users' Manual

1、Product features

The folding stretcher is made of aluminum alloy and has the characteristics of novel structure, light and convenient. It is a portable stretcher. The stretcher can be folded and stored after use, which is convenient for transportation and carrying.

2、Main purpose and scope

It can also be used in hospitals, military fields, stadiums and other venues to transfer patients.

3、Structure composition

The stretcher is composed of crossbeam, longitudinal folding hinge, surface cloth, safety belt, etc. the safety belt is optional. See figure 1

4、Basic parameters

The basic parameters of stretcher are shown in Table 1

Table 1 Basic parameters of folding stretcher

Unit: mm

No.	Model	Product Size			Folding Size		
		Length	Width	Height	Length	Width	Height
1	WSX-D1C	≥2000	≥500	≥120	975±20	170±20	85±20

Company code

5、Capacity: 250kg

6、Instructions

6.1 This product is out-of-the-box.

6.2 Product deployment: each of the two people hold one end, open the stretcher horizontally and vertically, and then use it.

6.3 Product folding: two people hold one side and fold the stretcher horizontally and vertically inward respectively.

7、 Maintenance and disinfection

7.1 Use a face cloth or soft brush, dip in a little water and neutral detergent, and then wipe with a dry face cloth.

7.2 Disinfection with neutral disinfectant spray or wipe, Iodine and phenol disinfectants should not be used.

7.3 Check every week and tighten the screws.

7.4 Add lubricating oil to the rotating parts every month.

8、 Precautions

8.1 in order to ensure the convenient and safe transfer of patients, it is recommended that the user unit configure a full-time stretcher operator, who shall be trained, and untrained personnel shall not operate the stretcher!

8.2 The stretcher must be operated by at least two persons. When rescuing the wounded, the safety belt must be fastened. It is strictly forbidden to operate the stretcher savagely or illegally, otherwise it may be damaged!

8.3 It is very necessary to carry out the operation drill with a stretcher that is without a man. Personnel without the above study and practice are not allowed to operate the stretcher!

8.4 This product is only suitable for carrying the sick and wounded. It is strictly prohibited to load or transport other objects overweight. It is strictly prohibited to collide with ambulance or other objects violently.

8.5 It is forbidden to wash with high pressure water gun.

8.6 Regularly check whether the bolts at all parts are loose and add lubricating oil at all moving parts.

8.7 If any parts are damaged, please contact our company. The use of non our company's elements will bring stretcher security risks!

8.8 The damage of stretcher elements, operation method and use environment may cause potential safety

hazards of stretcher. Once the stretcher may cause the patient to fall or be injured, the operator must hold the stretcher firmly before and after the stretcher to prevent the patient from being injured due to the fall of stretcher.

9、Contraindications: none.

10、During shipment, general transportation tools can be used or according to the order contract. During transportation, severe collision, sunlight and rain should be avoided. The packed folding stretcher should be stored in a well-ventilated room without corrosive gas to avoid mixing with toxic and harmful substances.

11、Contact information and after sales service

The warranty period of this product is one year, and the following situations do not belong to the warranty scope:

①Damage or failure caused by improper use.

②Damage or failure caused by failure to operate, maintain and maintain according to this manual.

③ Damage or failure caused by unforeseen natural disasters.

During the warranty period, the company provides maintenance parts for free. After the warranty period, the company is also responsible for the maintenance / replacement of products and charges the corresponding fees. Not responsible for other quality assurance responsibilities except warranty.

Company Name: Jiangyin Everise Medical Equipment Co., Ltd.

Registered address: 1001 Chengyang Road, Jiangyin City, Jiangsu Province, China

Production address: 1001 Chengyang Road, Jiangyin City, Jiangsu Province, China

Tel: 0510-86900900

Fax: 0510-86900111

Zip Code: 214423

We declare that the device complies with MDR 745/2017 "Medical Devices

It is necessary to report any serious accident occurring in relation to the DM provided by PVS SPA to the Manufacturer and the competent authorities

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